

Make USB Update USB Stick Instructions







Doc date: 7/11/13

Purpose: The purpose of these instructions are to give guidance on how to properly download an treatment file, add (copy) to USB drive and add treatments to the LIGHTWAVE/DK main unit using the USB port and USB flash drive.

Items Required: You will need the following items to perform the update.

1. LW DK Main Unit with USB port
2. Any flash drive
3. PC or Laptop (w/ USB port)
4. Email/Internet Access
5. Update File/Folder: LIGHTWAVE.zip = Folder "LIGHTWAVE/Update/LWUpdate.upd" (Compressed for email. Needs to be extracted)

Instructions:

<p>1. Locate the USB flash drive provided by LTW or any other USB flash drive available.</p> 	<p>2. Go to your email inbox and find the email with the treatment file attached. "LIGHTWAVE.zip"</p> 	<p>3. Download or save the attachment to your PCs "desktop".</p> 	<p>4. Insert USB flash drive into any USB port on your PC or Laptop.</p> 
<p>5. Open "Windows Explorer" and navigate to your desktop (or location of where you saved the file)</p> <p><i>NOTE: you can open Windows Explorer by pressing the Windows Key and "E" simultaneously.</i></p>	<p>6a. The file is a "zip" and needs to be extracted.</p> <p>How to extract zip file</p> <p>6b. Once extracted select (highlight) "LIGHTWAVE" folder and copy the folder to clipboard.</p> <p><i>NOTE: you can select by Right clicking on file. Ctrl+C = Copy or Right click and select</i></p>	<p>7. Open "Windows Explorer" and navigate to your USB Drive (typically drive E or F, but not always)</p> <p>7b. At the root of the USB drive, paste the extracted folder you copied.</p> <p>Your Windows Explorer navigation bar should display an address like: E:\LIGHTWAVE and the LWUpdate.upd file will reside in the UPDATE folder.</p> <p>7c. Double click on the LIGHTWAVE folder and you should see a folder named UPDATE with the file named "LWUpdate.upd" in it.</p> <p>If this is correct you can move on to the next step.</p>	
<p>8. At this time remove the USB flash drive from the USB port and proceed to the next step to add treatments to the LWDK system.</p> 	<p>9. Insert USB flash drive into USB port located on right side of main unit and re-boot the MU. During startup it will start the update process.</p> 	<p>10a. System will prompt you that "An update is available..." Select Yes</p> <p>10b. Once completed the system will finish starting up and bring you to the login screen.</p> <p>At this point the system has been updated and can be used as normal.</p>	

If you have any questions or need assistance with this process please contact LTW's support staff for help.

Thank you,

LTW Support Staff
www.myLIGHTWAVE.com
 866.999.6954 x 5